



## Volumetric incentive spirometers



Oxygen and Aerosol Therapy ▪ Respiratory Physiotherapy


## Volumetric incentive spirometer

The volumetric incentive spirometer helps patients perform and monitor their breathing exercises. The patients are encouraged to breathe in through the device as slowly and deeply as possible. The volumetric incentive spirometer provides visual guidance of the volume achieved during inspiration whilst encouraging the patient to maintain an appropriate inspiratory flow rate, by ensuring the yellow ball is kept within the smiley face section of the device.

### Compact and ergonomic design

#### Two size options available

5000ml for adult and 2500ml for paediatric

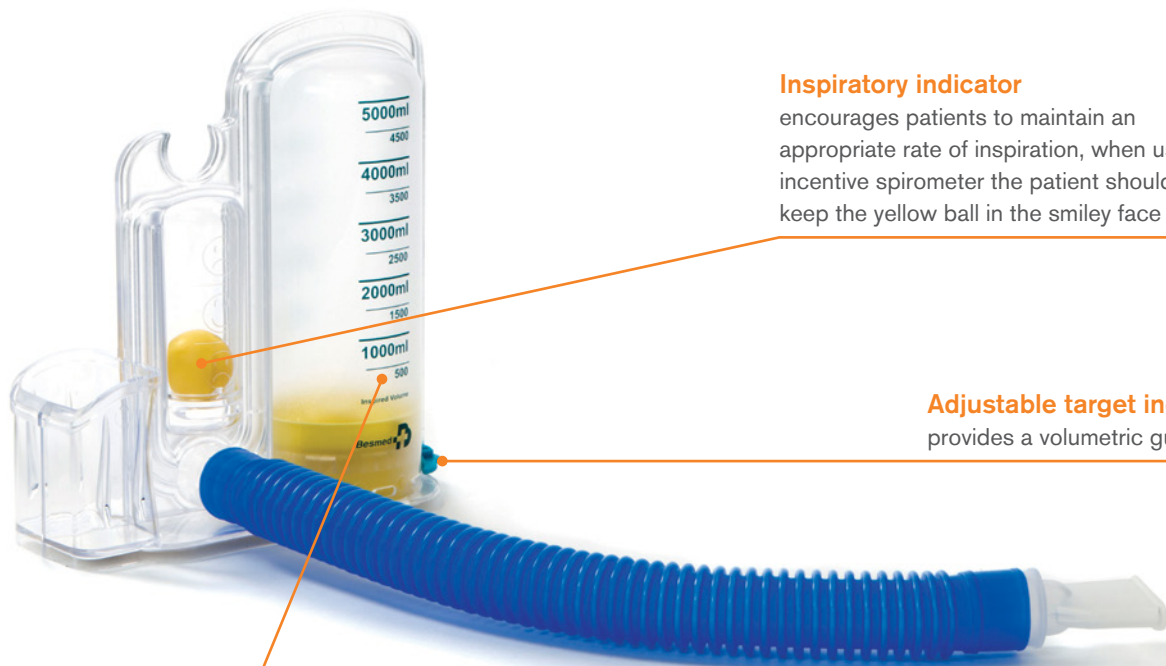

#### Inspiratory indicator

encourages patients to maintain an appropriate rate of inspiration, when using the incentive spirometer the patient should aim to keep the yellow ball in the smiley face section

#### Adjustable target indicator

provides a volumetric guideline

#### Volume scale printed on both sides

dual-sided, clear identification of volumes

Code	Description	Box Qty.
1535015	Volumetric incentive spirometer 5000ml (adult)	12
1536015	Volumetric incentive spirometer 2500ml (paediatric)	12

Make an enquiry

The Volumetric Incentive Spirometers are manufactured by Besmed Health Business Corporation, Taiwan and distributed by Intersurgical Ltd

Please think before you print Save energy and paper. If you must print this information sheet please print it double sided.

IS10.36 • Issue 2 01.21



Intersurgical Ltd, Crane House, Molly Millars Lane, Wokingham, Berkshire, RG41 2RZ, UK  
T: +44 (0)118 9656 300 F: +44 (0)118 9656 356 info@intersurgical.com www.intersurgical.com

UK • Ireland • France • Germany • Spain • Portugal • Italy • Benelux • Sweden • Denmark • Lithuania • Russia • Czech Republic  
Turkey • South Africa • China • Japan • Taiwan • Philippines • USA • Canada • Colombia • Australia