# KLFMED<sup>®</sup>

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**This instruction for use is not part of products and is issued on customer request**

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- COROSPIKE GREEN ref: CPHF-CS01
- COROSPIKE BLUE ref: CPHF-CS05
- COROSPIKE RED ref: CPHF-CS10

Read the instructions carefully.

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#### **Indications for Use:**

GREEN type: easy withdrawal and supplementary injection, reliable protection against microbial contamination by integrated air filter 1,2 µm.

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BLUE type: reliable protection against particles during withdrawal by 5 µm solution filter and with integrated air filter 1,2 µm

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RED type: reliable protection against particles during withdrawal by 5 µm solution filter and with integrated air-vent filter 0,2 µm which is reliable handling of very sensitive solution for contamination and by protection against toxic aerosols

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**Cautions:** Do not clean or re-sterilize. This device remains sterile until the package is opened or damaged. Do not use if the sterile package has been opened or damaged.

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**Re-sterilization:** Product is not intended for re-sterilization. Manufacturer prohibits re-sterilization.

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**Dangerous substances:** If unit package is marked with symbol "PHT-DEHP" product contains substances called dangerous. Corospiques do not contain dangerous substances.

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**How to use:** Fully tighten all connection before use. Use aseptic techniques.

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2. Put spike in bottle

**Vpich trn do lahve**

3. Open clips-vent by spike and connect syringe with spike

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4. Draw liquids through Luer-Lock connection on spike to syringe

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5. Disconnect syringe with spike and close clips-vent

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6. Re-connect syringe with spike according to medical/hospital rules.

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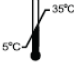
7. Destroy spike when disconnecting with bottle. Throw away.

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# **KLFMED**<sup>®</sup>

Used symbol on blister

 <b>REF</b>	Product reference		For single use only	 <b>STERILE EO</b>	Sterilized by Ethylene oxide
 <b>LOT</b>	Batch/Lot number		Do not use if unit package is damaged		
	Date of Manufacturing		Keep away from sunlight		
	Use by/Expiry date		Keep away from rain		
	Manufactured by		Pyrogen free		
	See instruction for use		Temperature limitation		